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**GRAZIE!**

**DISPOSAL OF THE ELECTRIC OR ELECTRONIC DEVICE AT THE END OF ITS LIFE**

**INFORMATION FOR USERS ABOUT THE CORRECT DISPOSAL  
OF THE PRODUCT ACCORDING TO THE EUROPEAN DIRECTIVE 2012/19/EU  
AND TO THE LEGISLATIVE DECREE N. 49/2014.**

**DEAR CUSTOMER,**

**IT IS IMPORTANT TO REMIND YOU THAT, AS A GOOD CITIZEN, YOU CANNOT DISPOSE OF THIS PRODUCT AS MUNICIPAL WASTE; YOU HAVE TO DO A SEPARATE COLLECTION. THE SYMBOL LABELLED ON THE APPLIANCE INDICATES THAT THE RUBBISH IS SUBJECT TO "SEPARATE COLLECTION". ELECTRICAL AND ELECTRONIC APPLIANCE CANNOT BE CONSIDERED AS DOMESTIC WASTE. THEREFORE, AT THE END OF ITS USEFUL LIFE, THIS PRODUCT SHOULD NOT BE DISPOSED WITH MUNICIPAL WASTES. THE USER MUST THEREFORE ASSIGN OR (HAVE COLLECTED) THE RUBBISH TO A TREATMENT FACILITY UNDER LOCAL ADMINISTRATION, OR HAND IT OVER TO THE RESELLER IN EXCHANGE FOR AN EQUIVALENT NEW PRODUCT.**

**THE SEPARATE COLLECTION OF THE RUBBISH AND THE SUCCESSIVE TREATMENT, RECYCLING AND DISPOSAL OPERATIONS PROMOTE PRODUCTION OF APPLIANCES MADE WITH RECYCLED MATERIALS AND REDUCE NEGATIVE EFFECTS ON HEALTH AND THE ENVIRONMENT CAUSED BY IMPROPER TREATMENT OF RUBBISH.**

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## ONLY FOR A DOMESTIC USE

ATTENTION! READ THE INSTRUCTIONS GIVEN IN THIS HANDBOOK VERY CAREFULLY.

IT SHOULD BE PUT ASIDE FOR FURTHER CONSULTATION DURING THE ENTIRE LIFE-CYCLE OF THE PRODUCT SINCE IT GIVES IMPORTANT INDICATIONS ON THE PRODUCT'S FUNCTIONS AND SAFETY DURING INSTALLATION, USE AND MAINTENANCE.

GALILEO SPA - STRADA GALLI N. 27 - 00019 VILLA ADRIANA (RM) - MADE IN EXTRA EU

## GENERAL SAFETY INDICATIONS

- DO NOT IMMERSE ANY PART OF THE DEVICE IN WATER
- DO NOT EXPOSE TO HEAT OR FIRE.

## DANGER OF EXPLOSION!

- DO NOT PERFORATE OR TAMPER WITH BATTERIES;
- DO NOT THROW BATTERIES INTO FIRE.
- DO NOT DISASSEMBLE, CUT, COMPRESS, BEND, PIERCE OR DAMAGE BATTERIES. IN CASE OF LEAK FLUID OF BATTERY, AVOID CONTACT OF THIS LIQUID WITH SKIN AND EYES. IN CASE OF ANY CONTACT, WASH THE AFFECTED AREA WITH A COPIOUS AMOUNT OF WATER OR CONSULT IMMEDIATELY YOUR DOCTOR.

DO NOT MODIFY, INSERT FOREIGN OBJECTS INTO BATTERIES AND DO NOT SOAK OR EXPOSE THEM TO WATER OR OTHER LIQUIDS. DON'T USE BATTERIES OR CHARGER FOR ANY PURPOSE EXCEPT ITS INTENDED ONE.

PLEASE NOTE THAT ANY MISUSE OR DISAPPROVED OR INCOMPATIBLE USE OF BATTERIES AND

CHARGER CAN LEAD TO FIRE HAZARD, DEFLA-GRATION OR OTHERS DAN- GERS FOR WHICH THE PRODUCER DOES NOT TAKE ANY RESPON-SABILITY AND DOES NOT PROVIDE GUARANTEES.

- NEVER USE DAMAGED BATTERIES OR CHAR-GER. USE CHARGER ONLY INDOORS.
- DO NOT DISPOSE OF BATTERIES IN YOUR HOU-SEHOLD WASTE. RESPECT ALL LOCAL LAWS. RECYCLE WHENEVER POSSIBLE.

### **DAMAGES AND / OR LEAKAGES**

- PLEASE NOTE THAT NON-COMPLIANCE WITH THE ABOVE MENTIONED WARNING INSTRUCTIONS MAY LEAD TO ELECTRIC SHOCK HAZARDS, DAMAGES / LIQUID LEAKAGES, OVERHEATING, EXPLOSIONS OR FIRES AND THUS CAUSING SERIOUS PERSONAL INJURIES AND / OR DAMAGES TO OBJECTS.

### **END-OF-LIFE BATTERIES/ACCUMULATORS TREATMENT**

- TREATMENT OF END-OF-LIFE BATTERIES / AC-CUMULATORS BATTERIES/ACCUMULATORS AND THE ELECTRIC OR ELECTRONIC DEVICE CANNOT BE DISPOSED AS DOMESTIC WASTE, THIS HAS TO BE DONE IN CONFORMITY WITH 2002/96/EC AND 2006/66/EC DIRECTIVES AND TO NATIONAL LAWS ON WASTE ELECTRICAL AND ELECTRONIC EQUIPMENT. A SEPARATE COLLECTION IS ESSENTIAL TO AVOID DAMAGES TO THE ENVIRONMENT. IN ORDER TO REMOVE THE BATTERY, PLEASE RE-FER TO THE SPECIFIC INSTRUCTIONS. PLEASE NOTE THAT FAILURE TO RESPECT WHAT STATED ABOVE IS SANCTIONED BY THE LAW.

## COMPONENTS

## REQUIREMENT OF PHONES

Android™ 4.4 or above

Apple 8.0 or above

## BEFORE USE

Please make sure that the electricity of the wristband is sufficient before your first use.

If the charge level of the device is too low to make it start, please charge it

## APP DOWNLOAD

This fitness watch works with WEARFIT APP, both for Android and Apple devices.

## DEVICE CONFIGURATION

1) After purchase, holding the device with the left hand, remove the central body of the product by gently pushing up the top cover with the index finger

of the right hand while, with the thumb of the same hand, gently pushing down the silicone bracelet on the side opposite the sensor (white dot).

2) Connect the device to a USB socket on a PC and leave charging for at least 4 hours.

3) Once the charging is complete, reassemble the device by performing the same steps in the reverse order.

4) In order to use your device, you must download the app from the aforementioned stores.

5) To connect your device and application, open your phone settings and activate connections with wireless devices.

6) Open the application; select the “Notification Center” menu, “connection management”, enable device search, select the relevant device.

7) Allow application notifications.

8) Follow the setup wizard steps.

9) After connecting the bracelet correctly, the time and date will be synchronized. It is advisable for convenience to configure alerts and set all alarms on the device directly from the app.

## AVAILABLE FUNCTIONS

The app can synchronize motion status, such as step counter, sleep monitor, heart rate monitor, blood pressure, etc.

- **Start page:** Display time, date and charging status
- **Smart Alarm Clock:** Set the clock in the app and the device will vibrate at the indicated time
- **Pedometer:** Count your steps in one day, automatically resets after 24 hours. You can view step history in the app.
- **Calories:** Count calorie consumption, counter automatically resets after 24 hours. You can view your calorie history in the app.
- **Distance:** Count the miles traveled, resets automatically after 24 hours. You can view step history in the app.
- **Heart rate monitor:** Switch to the Cardiac frequency monitor interface, the test will start automatically and the dates will be saved in the app.
- **Blood Pressure Monitor:** Switch to the Blood pressure monitor interface, the test will start automatically and the dates will be saved in the app.
- **Sleep Monitor:** Open the sleep detection function in Band, detects your sleep status over 24 hours. Save the dates in the app.

- **Sedentary Reminder:** In “Band Function Setting”, activate the Sedentary reminder function and set the time, the device will vibrate when accordingly.
- **Smart Anti-Loss:** In “Band Function Setting”, turn on Smart anti-loss function; the bracelet vibrates when the phone is out of reach of the phone.
- **Notification message:** After completing pairing, when the phone receives a message or a phone call, the bracelet illuminates and vibrates.
- **Incoming Call Reminder:** After completing pairing, when there is an incoming call, the bracelet will show the contact.
- **Shake for selfies:** Open this function in “Setting function Band”, then click “Shake for selfies” in the menu bar and shake the bracelet to take a selfie.
- Once you have configured personal data on the application (weight, height, gender, age, etc.) the device can act as a calorie counter activating the “sport” functions through the various menus accessible via touch sensor. Consumption is best calculated by setting the program closest to the sporting activity being performed.
- To scroll through the various functions directly from the device, tap the sensor (white circle) with a fingertip; to open the corresponding section or to confirm a function, place the fingertip over the sensor and hold the position for two seconds.

## • REGULARLY CHECK FOR APPLICATION UPDATES.

**• ATTENTION: THE DEVICE IS RESISTANT TO SPLASHED WATER, BUT IS NOT SUITABLE FOR WASHING, SHOWERS, BATHS, SWIMMING, DIVING OR OTHER WATER ACTIVITIES. THESE ACTIVITIES INVALIDATE THE WARRANTY CONDITIONS. WIPE WITH A DAMP CLOTH.**

## HOW TO USE THE DEVICE

Once you have connected the device correctly, time and date will be automatically synchronized.

The App will monitor your movements such as Pedometer, Sleep monitor, Heart rate monitor, Blood pressure monitor, etc.

## WARNINGS

Warnings about the device:

- Avoid exposing the device to heat sources or elevated temperature, such as vehicles parked in the sun.

In order to avoid possible damages, remove the device from the vehicle or keep it away from the direct sunlight, for example by storing it in its case.

- When not using the device for a long period of time, store it by respecting the proper temperature range, specified in the instruction manual provided with the packaging of the device.

**ATTENTION! THE DEVICE IS NOT A MEDICAL DEVICE AND IT MUST NOT BE USED FOR ANY DIAGNOSTIC OR SIMILAR PURPOSES!**

**Always consult your doctor before starting or modifying a training program.**

## HEALTH WARNINGS

- In case of pacemakers or any other medical electronic internal device, consult your doctor before using this fitness watch.
- The sensor panel will emit a green flashing light. Consult your doctor if you suffer from epilepsy or if you are sensitive to bright or flashing lights.
- Always consult your doctor before starting or modifying a training program.
- The device and the related data are intended only for recreational purposes and they must not be used neither for medical purposes, nor to diagnose, monitor, treat, cure or prevent from any disease.
- Please note that data related to Heart rate monitor and Blood pressure are indicative; therefore it is not admitted any responsibility in case of any eventual misreading.
- Although the optical monitoring technology of the heart rate frequency provides generally a precise estimate of the individual heart rate, in some cases there may occur technological limits which may lead to an imprecise interpretation of the resulting data. These is the case of physical characteristics, device suitability and the kind and intensity of the activity performed.
- Even if the pulse oximeter sensor technology provides a precise estimate of the blood pressure level, in some cases there are some technological

limitations that may lead to a misinterpretation of the data, especially as for those ones concerning physical characteristics of the individual, the device suitability, the device location and movement sensors.

- The activity trackers are based on sensors which monitor movements and other parameters. Data and information given by those devices provide a realistic estimate of the activity and recorded metrics but note that they are not 100% accurate. This concerns data related to sleep tracking data, steps, distance, heart rate frequency and calories burned.

Android, Google Play, and the Google Play logo are trademarks of Google LLC. Apple, iPhone, iPod touch, and iTunes is a trademark of Apple Inc., registered in the U.S. and other countries.

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Galileo S.p.A