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Grazie!

HONSUN (NANTONG) CO., LTD.

Address: No.8, Tongxing Road, Economic & Technical Development Area, Nantong City, Jiangsu, P.R.China

Shanghai International Holding Corp. GmbH (EUROPE)
Address: Eiffeustrasse 80, 20537 Hamburg, GERMANY

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13. PERIODIC SAFETY CHECKS 5

14. SPECIFICATIONS 6

15. GUIDANCE AND MANUFACTURER'S DECLARATION 6

16. QUALITY GUARANTEE 6

17. REQUIRING RECORD 6

INSTRUCTION

Thank you for purchasing the NB-221C Compact Compressor Nebulizer.

This product was developed for the successful treatment of asthma, allergies and other respiratory disorders. The compressor forces air to the nebulizer. When the air enters the nebulizer, it converts the prescribed medication into an aerosol of microscopic droplets that can easily be inhaled.

SYMBOLS

Symbols	Meaning
	Manufacturer
	Authorized Representative in the European community
	Symbol for the marking of electrical and electronics devices according to Directive 2002/96/EC. The device, accessories and the packaging have to be disposed of waste correctly at the end of the usage. Please follow Local Ordinances or Regulations for disposal.
	CE marking in conformity with EC
	Keep dry
	Attention, consult
	Type BF Applied Part

SAFETY INFORMATION

To assure the correct use of the product, basic safety measures should always be followed including the warnings and cautions listed in this instruction manual.

WARNING

- For regime of medication shall follow the instructions of your physician or licensed healthcare practitioner.
- Do not cover the compressor with a blanket, towel, or any other type of cover during using. This could result in the compressor overheating or malfunctioning.
- Do not use the device where the device may be exposed to flammable gas or vapors.
- Always dispose of any remaining medication in the medication tank after each use. Use fresh medication each time you use the device.
- Do not leave the device or its parts where it will be exposed to extreme temperatures or changes in humidity, such as leaving the device in a vehicle during warm or hot months, or where it will be exposed to direct sunlight.

CAUTION

- Limit the use of the device to 20 minutes at a time, and wait 40 minutes before using the device again.
- Provide close supervision when this device is used by, on, or near infants, children or compromised individuals.
- Do not insert any object into the compressor.
- Make sure that the air filter is clean. If the air filter has changed color or has not been used for 60 days, replace the filter.
- Make sure that the nebulizer kit is correctly assembled, the air filter is properly installed, and the air tube is correctly connected to the compressor and the nebulizer kit. Air may leak from the air tube during use if not securely connected.
- Inspect the compressor (main unit) and the nebulizer parts each time before using the device. Make sure no parts are damaged, the nozzle and air tube are not blocked and the compressor operates normally.
- Do not use the device if the air tube is bent.
- Do not block the air filter cover.
- Do not alter the baffle, the nozzle in the medication tank or any part of the nebulizer kit.
- Do not add more than 10ml of medication to the medication tank.

CAUTION

- Do not operate the device at temperatures greater than 40°C.
- Do not tilt the nebulizer kit so the angle of the kit is greater than 45°. Medication may flow into the mouth.
- Do not shake the nebulizer kit while using the device.
- Do not subject the compressor, or any of the components to strong shocks, such as dropping on the floor.
- Do not disassemble or attempt to repair the device or components.
- Use the device only for its intended use as described in the instruction manual. Do not use attachments not recommended by the manufacturer.
- Dispose of the device, components and optional accessories according to applicable local regulations.
- Make sure that the air tube is securely attached to the compressor (main unit) and nebulizing parts, and does not come loose. Twist the air tube slightly when inserting it into the connectors to avoid the tube disconnecting during use.

RISK OF ELECTRICAL SHOCK


- Do not use the compressor (main unit) and the power cord while they are wet.
- Do not plug or unplug the power cord into the electrical outlet with wet hands.
- Do not immerse the compressor (main unit) in water or other liquid.
- Do not spill water or other liquids on the compressor. These parts are not waterproof. If liquid spills on these parts, please unplug the power cord and wipe off the liquid with gauze or other soft absorbent material immediately.
- Do not use or store the device in humid locations or outdoors. Use the device within the operating temperature and humidity.
- Do not overload power outlets. Plug the device into the appropriate voltage outlet.
- Do not use extension cords. Plug the power cord directly into the electrical outlet.
- Unplug the power cord from the electrical outlet after using the device. Never leave this product unattended when plugged in.
- Unplug the power cord from the electrical outlet before cleaning the device.
- Completely read all of the instructions included the optional accessories before using them.

MAINTENANCE AND STORAGE

- Keep the device out of the reach of unsupervised infants and children. The device may contain small parts that can be swallowed.
- Do not leave the cleaning solution in the nebulizer parts. Rinse the nebulizer parts with clean hot tap water after disinfecting.
- Wash the nebulizer parts after each use. Dry the parts immediately after washing.
- Do not store the air tube with moisture or medication remaining in the air tube. This could result in infection as a result of bacteria.
- Store the device and the components in a clean, safe location.
- Do not carry or leave the nebulizer with medication in the medication tank.

- Do not place or attempt to dry the device, components or any of the nebulizer parts in a microwave oven.
- Do not wrap the power cord around the compressor (main unit).

MAIN UNIT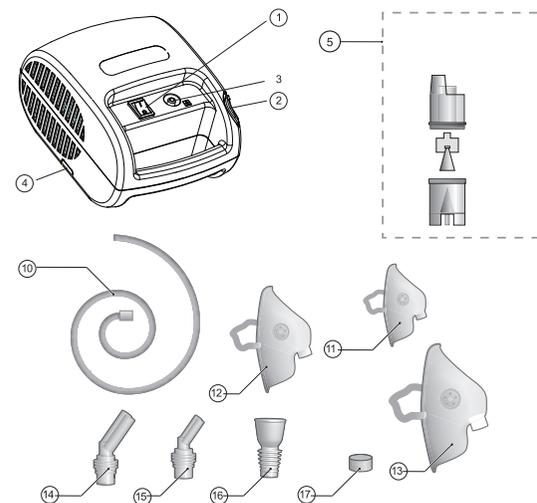

- 1. Power switch
- 2. Holder for Nebulizer
- 3. Air plug
- 4. Dust Screen Cover
- 5. Nebulizer Kit
- 6. Handle
- 7. Main Unit
- 8. thermal Via
- 9. Rubber Foot
- 10. Inhalation Tube
- 11. Small Child Mask(Optional)
- 12. Child Mask(Optional)
- 13. Adult Mask
- 14. Adult Nasal-piece(Optional)
- 15. Child Nasal-pieces(Optional)
- 16. Inhalation Mouthpiece
- 17. Air Filter x 4

PREPARING THE NEBULIZER FOR USE

CAUTION

- Clean and disinfect the nebulizer kit and optional masks before using them for the first time after purchase.
- If the device has not been used for a long period of time, please cleans and disinfects the nebulizer kit and optional masks before using them.
- To prevent the risk of infections, the nebulizer kit should be sterilized before using from different persons.

- At normal use, place the device on a stable, sturdy and flat surface horizontally, such that the unit can be easily reached when you are seated.

Caution: place the device at least 10cm distance from walls.

- Pull out the power cord from the base of the unit, and make sure that the unit is in the "off" (0) position by pressing on the right side of switch.
- Insert the power plug into the electrical outlet.

WARNING

Power Switch is not a safety disconnecting switch, the MAINS plug is intended to be used as the disconnect device. Please remain plug readily operable. Use of plug is as the intended isolation means.


- Rotate the inhalation top counterclockwise to remove the inhalation top from the medication tank.
- Add the correct amount of prescribed medication to the medication tank.
- Turn the inhalation top clockwise until securely closed
- Attach the desired inhalation accessory.

ATTACHING THE AIR TUBE

- Push the Air Plug on one end of the Air Tube onto the Air Connector on the front side of the compressor.
- Push the Air Plug on the other end of the Air Tube onto the Air Tube Connector on the bottom of the Nebulizer Kit.

USING THE DEVICE

- Take out of the power cord from the back of device and switch on the power.
- Plug the inhalation tube into the air plug under cover of device.
- Plug the other side of tube into the vent port of bottom of nebulizer kit.
- Plug the nebulizer kit into the holder on the device.
- Add the medicine into the nebulization tank.
- Open the O/I button.
- The device begins to nebulizer.

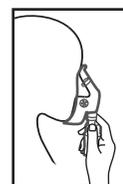CAUTION

Unplug the power cord from the electrical outlet after using the device.

USING THE CHILD MASK or THE ADULT MASK

Place the mask over the nose and mouth. Pull the elastic strap over the head. Gently pull on the strap to securely hold the mask over the nose and mouth. Inhale the medication. Exhale normally through the mask.

CLEANING AND DISINFECTING

CLEANING AFTER EACH USE

Following the cleaning instructions after each use will prevent any remaining medication in the bottle from drying resulting in the device not nebulizing effectively and will help prevent infections.

WARNING

Wash the nebulizer parts after each use. Dry the parts immediately after washing.

1. Remove the inhalation accessory (mask, mouthpiece) from the nebulizer kit.
2. Disconnect the air tubing from the nebulizer.
3. Gently twist the inhalation top counterclockwise and lift to separate the nebulizer into two sections.
4. Remove the baffle.
5. Discard remaining medication in the medication tank.
6. Rinse all the parts of the nebulizer kit, the mouthpiece or mask in warm water and a mild detergent. Rinse the mask with hot tap water.
7. Hand dry or air dry in a clean environment using a soft, clean lint-free cloth.
8. Assemble the nebulizer and store the nebulizer kit in a dry, sealed bag.

DAILY DISINFECTING

You can disinfect daily by soaking the parts in a disinfectant solution. If your physician or respiratory therapist specifies a different cleaning procedure follow their instructions. Disinfect the nebulizer kit and the mask or mouthpiece after the last treatment of the day.

1. Disconnect the all parts according to the above 1-6 steps.
2. Make enough solution to submerge the parts.
3. Submerge the parts in the white vinegar and distilled water solution for 30 minutes.
4. Remove the part and discard the solution. Rinse the parts with hot tap water.
5. Hand dry or air dry in a clean environment using a soft, clean lint-free cloth.
6. Assemble the nebulizer and store the nebulizer kit in a dry, sealed bag.

CARING FOR THE DEVICE

To keep your device in the best condition and protect the unit from damage follow these directions:

CLEANING THE DEVICE

Clean the casing of the main unit by using a soft cloth moistened with water or a mild detergent. Do not use abrasive cleaners. Dry the casing immediately using a soft clean cloth.

CHANGE THE AIR FILTER

Change the air filter every 60 days even if the air filter does not appear dirty. If the air filter appears dirty, or if water or medication has spilled on the air filter, replace with a new air filter immediately.

1. Pull the air filter cover to remove from the front side of the compressor.
2. Remove the dirty filter with hand.

CAUTION

Do not attempt to wash or clean the air filter. Damp air filters can cause blockages. Do not substitute cotton or any other material for the air filter.

CAUTION

Wash the air filter cover regularly to prevent any blockage in the cover. Do not boil. Make sure the cover is dry before inserting the new air filter.

3. Insert a new air filter into the air filter cover.

CAUTION

Before inserting the new air filter makes sure the air filter is clean and free of dust. Do not operate the device without the air filter.

4. Put the air filter cover back on the compressor.

STORING THE DEVICE

1. Put the nebulizer kit and the inhalation accessory (mouthpiece, or mask) in a dry, sealed bag.
2. Store the device in a safe, clean location. Do not store the device in extreme hot or cold temperature, high humidity or in direct sunlight.

WARNING

Do not leave the device or its parts where it will be exposed to extreme temperatures or changes in humidity, such as leaving the device in a vehicle during warm or hot months, or where it will be exposed to direct sunlight.

WARNING

Keep the device out of the reach of unsupervised infants and children. The device may contain small parts that can be swallowed.

CAUTION

Do not carry or leave the nebulizer with medication in the medication tank.

Do not disassemble or attempt to repair the device or components.

TROUBLESHOOTING

TROUBLESHOOTING GUIDE

PROBLEM	CAUSE	SOLUTION
No power on unit when the power switch is on.	The AC power cord is not plugged into an electrical outlet.	Turn the power switch off. Plug the power plug into an electrical outlet. Turn the device on.
No nebulization or low nebulization rate when the power is on.	No medication in the medication tank. Too much or too little medication in the medication tank.	Add the correct amount of prescribed medication to the medication tank.
	The nebulizer kit is not correctly assembled.	Make sure the nebulizer kit is correctly assembled and the inhalation accessory is correctly attached.
	The nebulizer kit is tilted at an incorrect angle.	Hold the nebulizer kit correctly. Do not tilt the nebulizer kit so the angle of the kit is greater than 45 degrees.
The device is very hot.	The air tube is incorrectly attached.	Make sure the air tube is correctly attached to the compressor and the nebulizer kit.
	The air tube is folded or damaged. The air tube is blocked.	Make sure the air tube is not folded, kinked or bent. Inspect the air tube for any damage. Replace the air tube if damaged.
	The compressor is covered. The device has been used for longer than 20 minutes.	Do not cover the compressor with any type of cover during use. Turn the device off. Wait 40 minutes before using the device again.

PERIODIC SAFETY CHECKS

Preventive inspection and maintenance to be performed including the frequency of such maintenance

1. Please clean the plug of power cord at least once a year. Too much dust on plug may cause the fire.
- 2) Inspect the equipment and accessories for mechanical and functional damage.
- 3) Inspect the safety relevant labels for legibility.
- 4) Verify that the device functions properly as described in the instructions for use.

SPECIFICATIONS

Model:	NB-221C
Rated Voltage:	AC 230V, 50HZ
Medical Device:	Class II, Type B
Nebulizing Pressure:	100~190Kpa
Nebulizing Flow:	4~7L/min
Extreme Pressure:	>205kpa
Rated Power:	60W
Max. Capacity of Medication Tank:	10ml
Particle Size:	≤5µm
Nebulizing Rate:	≥0.2ml/min
Noise Level:	≤65db
Operating Temperature:	10 C- 40 C
Size:	140mm x 90mm x 190mm

COMPRESSOR NEBULIZER With respect to electric shock, fire and mechanical hazards only in accordance with IEC60601-1.

Guidance and manufacture's declaration – electromagnetic emissions-for all EQUIPMENT and SYSTEMS

Guidance and manufacture's declaration – electromagnetic emission

The NB-221C is intended for use in the electromagnetic environment specified below. The customer of the user of the NB-221C should assure that it is used in such environment.

Emission test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The NB-221C uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emission CISPR 11	Class B	The NB-221C is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/flicker emissions IEC 61000-3-3	Complies	

DISPOSAL OF THE ELECTRIC OR ELECTRONIC DEVICE AT THE END OF ITS LIFE

Information for users about the correct disposal of the product according to the European Directive 2012/19/EU and to the Legislative Decree N. 49/2014.

Dear Customer,

It is important to remind you that, as a good citizen, you cannot dispose of this product as municipal waste; you have to do a separate collection.

The symbol labelled on the appliance indicates that the rubbish is subject to "separate collection". Electrical and electronic appliance cannot be considered as domestic waste. Therefore, at the end of its useful life, this product should not be disposed with municipal wastes. The user must therefore assign or (have collected) the rubbish to a treatment facility under local administration, or hand it over to the reseller in exchange for an equivalent new product.

The separate collection of the rubbish and the successive treatment, recycling and disposal operations promote production of appliances made with recycled materials and reduce negative effects on health and the environment caused by improper treatment of rubbish.

Thank you!

HONSUN (NANTONG) CO.,LTD.

Address:No.8, Tongxing Road, Economic&Technical Development Area, Nantong City, Jiangsu, P.R.China

Shanghai International Holding Corp. GmbH (EUROPE)
Address: Eiffestrasse 80, 20537 Hamburg, GERMANY

The manufacturer reserves the right to make technical changes without notice in the interest of progress. Prior notices will not be given in case of any amendments within this manual. The mentioned trademarks and names are owned by the corresponding companies.